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## **Directions for use of methatec- Cupping System with valve**

**Dear** customer, we congratulate to the acquisition of a cupping glass with or without the blue vacuum suction pump with integrated valve function.

This system is defined by:

- Individual vacuum setting
- High vacuum strength
- Robustness and
- a significantly easier cleaning in comparison to traditional cupping glasses.

Our cupping glasses are made out of a high quality laboratory glass and are handmade manufactured. Therefore there may be some minor tolerances in size.

Our cupping glasses are dedicated for massage and wellness applications.

### **Warnings**

**Please read diligently and completely directions for use before use and follow the cleaning instructions.**

---

Note: read Directions for use before operating!

### **Intended use**

For general massage applications and non-bloody cupping therapy for external application (skin). Applications should be performed by professionals in the mentioned areas of expertise.

### **Operation**

Place glass on the skin and press suction pump so often until you reach desired vacuum strength. If vacuum becomes weaker with time, it can be easily renewed by pressing again, while the cupping glass is still in place.

Take care that the glass circumference sits completely on the skin. ( choose correct size for selected sin area )

Hairy skin may make it difficult to reach good vacuum. In this case use massage oil or shave the hairs.

Removing the glass, press skin a bit and lift the glass carefully sidewise and vacuum is releasing.

## **Non-intended use / Contraindications**

The manufacturer is assuming no liability for damages, injuries or other incidents, which are a result of a non-intended use. The risk is carried completely by the operator. Pay attention that the glass is not exposed to any shock. Risk of breakage !

## **Handling and storage**

Handling and storage of the cupping instrument has to be in a dust and moisture free and recontamination protected area.

## **Cleaning**

The cupping glasses are made out of high quality laboratory glass and can be cleaned and disinfected after use (i.e. with Braun Helipur) .

The high quality glass is also autoclavable and can be sterilized as well.

Due to the big fitting diameter of the glass with the suction ball, cleaning is significantly easier and more safe than with systems where the suction pump is stucked to a small diameter nozzle. The blue suction pump can be wipe - cleaned and wipe - disinfected after removal from glass. (i.e.with Bode Microbac Tissues ) – not suitable for spray and immersion cleaning and disinfection.

## **Materials**

The blue vacuum suction ball is made of high quality PVC-Plastisol-soft, health safe and free of Cadmium or Latex. The material can be recycled. Avoid to use alcohol or oils - those can dissolve the softener and pump may be solidify.

## **Disposal:**

The product can be disposed by household waste.

**Hersteller:** methatec® GmbH & CoKG \*  
Gerlenhoferstr.4 \* 89233 Neu-Ulm \*  
Tel: +49 (0)7307 97550 \* Fax: +49 (0)7307 9755 30 \*  
[www.methatec.de](http://www.methatec.de) \* [vertrieb@methatec.de](mailto:vertrieb@methatec.de)

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**Fabricante:** methatec® GmbH & CoKG \*